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**Intended Use:**

The walkers can be used by people with limited mobility for travel within and outside the home.

**Assembling:**

The walkers are delivered assembled & ready to use.

**Further Instructions:**

1. To fold or unfold the walker, use the handles or the button (if available)
2. When using the walker for sitting or standing up push the handles down, securing that the walker will stand still and not slip.
3. When using the walker must rely on both handles equally. Leaning only to one handle (without escort) can damage the handle or & cause a fall.

**Storage & Transportation:**

Close and fold the walker using the handles or button before storing or transporting it

**Maintenance & Cleaning:**

- Clean each section of the walker frequently.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
- If you need to disinfect the product, use a common, mild disinfectant.

**CAUTION (!):**

In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.

**IMPORTANT (!)**

Before using the walker **MAKE SURE** that:

- The walker is in full extension and is locked in this position.
- The wheels (if available) are firmly fixed
- The handles are the same height and the screws that secure them on the frame are tight.
- There are no cracks or fractures in the frame.
- Regularly clean the walker.
- Do not let the walker wet.
- Do not let children near the walker without supervision

**Environmental Protection:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

**Declaration of Conformity:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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**INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

**WARRANTY:**

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT cover the parts, like brakes or rubber tips, that can wear out during use or over time.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user-customer or by dealer.

**WARRANTY FORM:**

BUYERS DETAILS:	
FULL NAME:	
PURCHASING DATE:	
PHONE:	
LOT :	
DEALER DETAILS	
FULL NAME:	
ADDRESS:	
PHONE:	
<b>SIGN AND SIGNATURE</b>	

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**MOBIAK®**  
MEDICAL • GASES • FIREFIGHTING est. 1977

**REF** : 0807998i, 0804458

**REF** :0809396i, 0804460

**REF** :0811606

MOBIAK S.A  
KAGIANA AKROTIRIOU, XANIA - KPTH  
KATHIANA AKROTIRIOU CHANIA - CRETE  
T: +30 28210 63222 | WEB: www.mobiaicare.com