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Revised:15/06

Medical Oxygen Flowmeter Diaphragm type

Manual of “APHRODITE”

IMPORTANT!

PLEASE READ THOROUGHLY AND FOLLOW THE DIRECTIONS CAREFULLY BEFORE OPERATING THE EQUIPMENT.

Specifications of “APHRODITE”

APHRODITE (Mobiak Model)

Medical Oxygen Flowmeter Click type O2

- Output pressure: 50psi
- CGA992 inlet connection, 0-15 LPM 20°C
- Bar pressure gauge, maximum 300 bar
- Metal flowmeter connector
- CE 0653
- APHRODITE Label (MOBIAK labelling) (one)
- APHRODITE Package (MOBIAK packaging)

WARNING!

Oxygen is a non-flammable gas; however, oxygen substantially increases its risk to become flammable when combined with oil, grease and other hydrocarbons. These materials should never be used or applied on any part of the oxygen cylinder, cylinder valve or other equipment used with oxygen.

GENERAL SAFETY INFORMATION

1. These regulators are intended for the administration of oxygen to patients that are deemed by a physician to need increased oxygen levels to improve or stabilize their breathing conditions.
2. Oxygen in a non-flammable gas; however, all materials which burn in the air will burn MUCH more rapidly in the presence of oxygen. OIL AND/OR GREASE BECOME HIGHLY COMBUSTIBLE IN THE PRESENCE OF OXYGEN! USE NO OIL or grease or any other petroleum based or flammable substance on or around oxygen equipment!

3. Standard industry cautions should be exercised when used in other applicable situations. These regulators may also be used in emergency situations (some models do not reach flows consistent with emergency applications). In any situation requiring emergency oxygen, call a physician or Emergency Medical Service technician immediately. This unit is an inhaler, and is useful only on persons who are breathing prior to administering oxygen.

4. If using a humidifier or a similar device, remove the device from the regulator before changing the cylinder. **DO NOT** allow the fluids to enter. **NEVER** attempt to remove a regulator or a humidifier from a cylinder if the pressure gauge(s) show pressure! **DO NOT** attempt to move the cylinder unless the regulator or the humidifier has been removed and the cylinder cap (when applicable) is in place.


5. Inspect the cylinder valve and the regulator thoroughly for dust, oil and grease. Wipe dusty parts with a damp cloth. **DO NOT USE THE CYLINDER IF OIL OR GREASE IS PRESENT!** Inform your gas supplier of this condition immediately.

6. Oxygen therapy and emergency oxygen regulators are pressure reducing devices which lower the pressure of the oxygen from a cylinder to a level which can be safely used.

7. **NO SMOKING** – Remove matches, cigarettes, lighters and lighter fluids from the patient and from the oxygen therapy area before administering oxygen. Remove other flammable materials from the area.

8. **DO NOT USE THE OXYGEN REGULATOR IF OIL, GREASE OR ANY CONTAMINATION IS PRESENT OR IF DAMAGE IS PRESENT!** Take the regulator to a qualified repair technician for cleaning and/or repairing before use. Inspect the regulator inlet filter in the inlet connection to be sure it is clean. If it appears dirty, take it to a qualified repair technician for replacement.

INLET CONNECTION DESCRIPTION

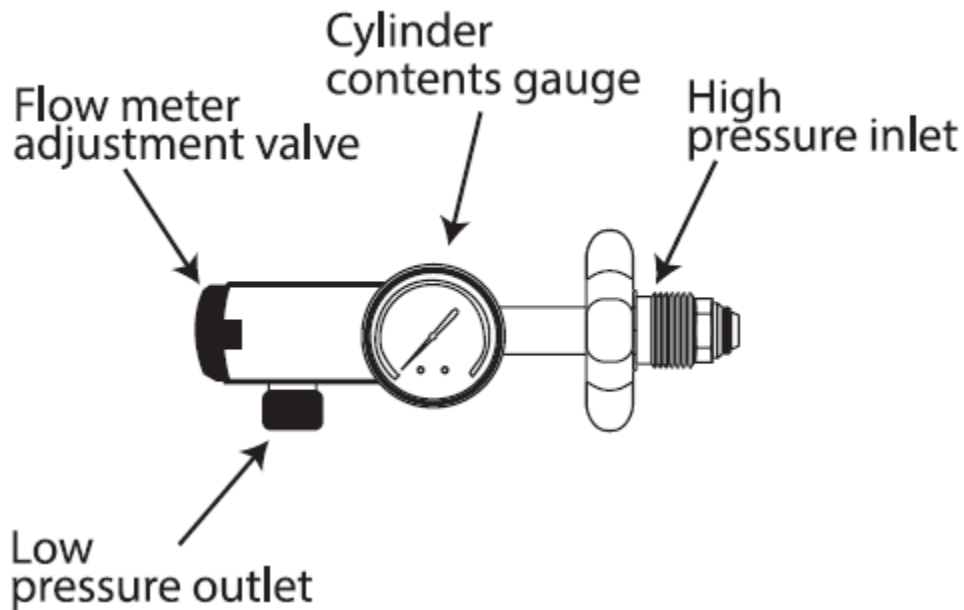- Output pressure: 50 psi
- CGA992 inlet, side entry, for clockwise female oxygen valve
- Bar pressure gauge: 0-300 bar
- **CAUTION:** Ensure that the sealing washer is in place on the inlet and secured to the valve. Tighten it manually using the plastic hand-wheel. Tighten the nut with a suitable wrench (N° 24) until secure to provide a leak-proof connection. With the valve outlet facing away from you, slightly open the oxygen cylinder post valve to purge any undetectable debris from the valve seat, then re-close the valve.

PRESSURIZING THE REGULATOR-FLOW SETTING

For a diaphragm type regulator, the output pressure can be adjusted. Turn the flow adjusting knob counter-clockwise (on) until the center of the ball corresponds with the line indicating the rate of flow prescribed by your physician (0-15 LPM 20°C). Allow the gas pressure in the regulator to completely escape. The flow gauge and high-pressure gauge needle should come to rest against the stop pin when all gas has escaped from the regulator.

WARNING

Do not use high concentrations of oxygen for more than 5 hours without 1-hour interruptions.

WARNING

Do not adjust the output pressure without training.

REGULATOR LEAK TESTING

If the regulator is leaking and must be moved from service and returned for repair, remember to always turn the cylinder valve fully off before removing or loosening the regulator connection.

WARNING

The maximum and minimum operating service temperatures should be no greater than 140°F (60°C) and no less than 0°F (-18°C).

ADJUSTING THE OXYGEN FLOW

Before opening the oxygen cylinder valve, be sure that the flow adjusting knob of the regulator is in the off position. This can be accomplished by turning the adjusting knob all the way out clockwise. For flowmeter regulators, turn the flow adjusting knob on the flow meter clockwise (off) until snug.

CLOSING THE REGULATOR

1. With the oxygen still flowing, turn the cylinder valve or wrench clockwise until snug.
2. Allow the gas pressure in the regulator to completely escape.
3. After all pressure has been relieved, turn the regulator to the off position.

CAUTION

- Do not use or store near heat or flame
- Do not puncture the cylinder in any way
- Do not attempt to remove the cylinder valve.
- Never throw the cylinder into fire or incinerate.

REMOVING THE REGULATOR FROM A CYLINDER

WARNING

NEVER attempt to remove a regulator from a cylinder if the pressure gauge(s) show pressure! **DO NOT** attempt to move the cylinder unless the regulator has been removed and the cylinder cap (when applicable) is in place. It is not necessary to remove the regulator unless the cylinder is being exchanged for a new one. A cylinder with less than 20 bar (300 PSI) showing on the high-pressure gauge should be exchanged for a full cylinder to ensure an adequate supply of

oxygen. Be sure that the cylinder valve is closed. Open the regulator momentarily to relieve all pressure before removing it from the cylinder. Remove the regulator from the cylinder by loosening the inlet connection counter-clockwise.

NOTE: Backpressure in excess of 0.15 bar (2 PSI) will cause inaccuracy of the delivery flows that are indicated on the adjusting knob. Common causes of backpressure are that the hose is kinked, the hose is too long between the user and the regulator, or any barriers restrict the flow occurring in that area. Don't cover or block the vent hole in the regulator body with labels, decals, or other adhesive material. Blocking vent may inadvertently affect the regulator's preset pressure, resulting in inaccurate flow rates.

ENVIROMENTAL PROTECTION

If you find one day that your product needs replacement, or no longer serves you, consider the environment:

- 1) Do not dispose of your product along with the municipal waste (this is also the importance of the stated recycling symbol).
- 2) Contact your municipal authority to indicate the point of disposal of your device for recycling.
- 3) You have to dispose of your useless product in the right place

Recycling helps to protect the environment and the operation again your product materials.

Warranty Form

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :

Address:.....

Phone:

Reseller Stamp

MOBIAC S.A
Kathiana- Akrotiriou
Chania-Crete, Greece
+30-28210-63222
www.mobiakcare.com