AEROpart® - WF 11

REF

HWF11

LOT

XXXXXX

QTY: 1

de	Wasserfalle für die Sauerstofftherapie	pl	Kaskada do terapii tlenowej
en	Water Trap for oxygen therapy	cs	Vodní kapsa pro kyslíkovou terapii
fr	Piège à eau pour l'oxygénothérapie	sk	Zachytávač vody pre kyslíkovú terapiu
es	Sifón de agua para tratamiento con oxígeno	no	Vannfelle for oksygenbehandling
it	Sifone per ossigenoterapia	sv	Vattenfälla för syrebehandling
pt	Separador de água para a oxigenoterapia	el	Παγίδα νερού για οξυγονοθεραπεία
nl	Waterval voor zuurstoftherapie		

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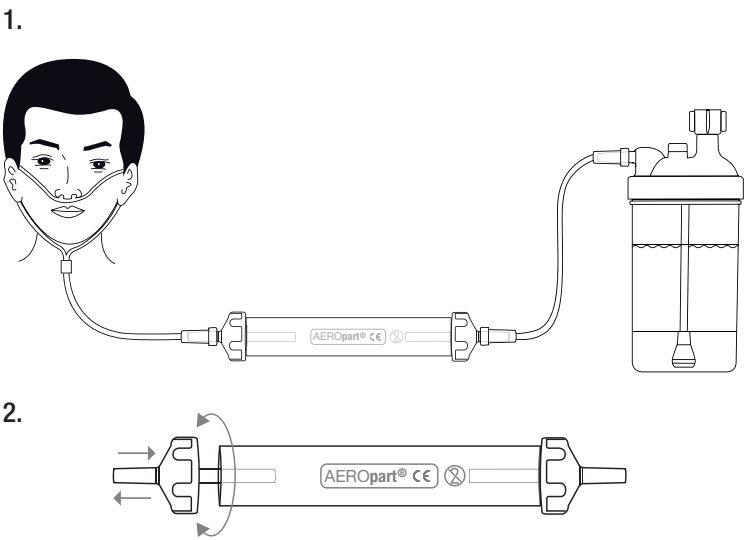
EC REP
mdi Europa GmbH
Langenhagener Str. 71,
D-30855 Langenhagen

CH REP
MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug Switzerland

HSINER Co.,Ltd.
No. 312, Jhongshan Rd., Shengang
Dist., Taichung City 429,Taiwan

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en Instructions for use

- 1. Intended purpose**
Water trap for oxygen therapy for collecting condensed water from the oxygen supply system.
- 2. Safety notes/Warnings:**
- Only to be used by trained and qualified personnel.
 - Disposable product for use on a single patient. Do not reuse. Reuse may cause cross infection, degradation of performance and/or product malfunction.
 - Open sources of ignition and smoking are strictly prohibited when operating oxygen therapy equipment. Observe the usual safety measures for handling oxygen.
 - Be sure to empty the water trap regularly to prevent the water from getting into the oxygen supply system.
 - If you detect that the product is damaged (eg breakage, cracks etc.) or leaking before or during use, please discard the product and replace it with a new one.
- 3. Operation**
- Connect the water trap between the bubble humidifier and the oxygen interface (eg nasal cannula or oxygen mask) to the oxygen tubes. Ensure that the connections are tight and do not have gas leaks. (Fig. 1)
 - To empty the water trap, unscrew one of the two end caps from the tube by a short counterclockwise rotation and dispose of the water. Then screw the end cap back onto the tube with a short turn clockwise. Ensure that the end caps fit tight and do not have gas leaks. (Fig. 2)

4. Disposal		
Dispose of according to the local relevant standards, regulations and laws after use.		
5. Technical data		
Operation temperature:	5 °C - 40 °C	Materials
Storage/Transport		End caps: Polyethylene
Temperature:	15 °C - 25 °C	Tube: Polycarbonate
Relative humidity:	< 85 %	Measures
	Store in a dry and clean place.	Length: 188 mm
		Diameter: 29 mm
		Weight: 22 g

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