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INTENDED USE:

The beds 0805420 & 0805421 are used for patients who have health problems, and they need to stay lying in bed.

They are designed for people who need to be treated for health reasons in institutions, hospitals, health care facilities or in a homecare environment.





CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts

PICTURES OF CORRECT AND WRONG USE OF THE PRODUCT:

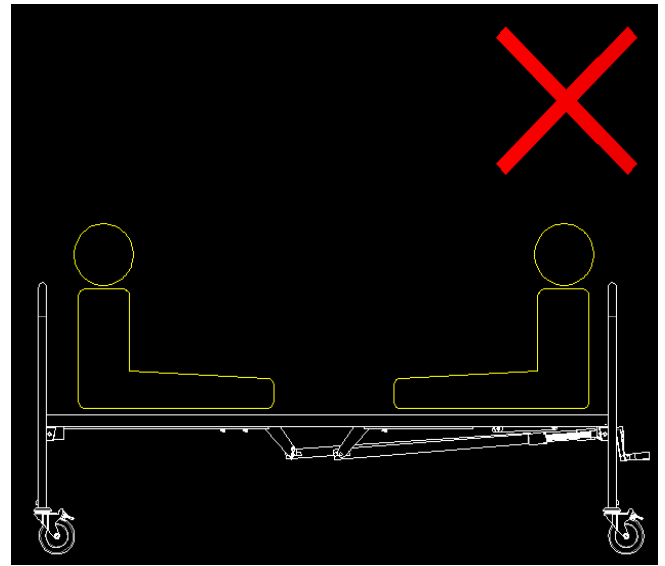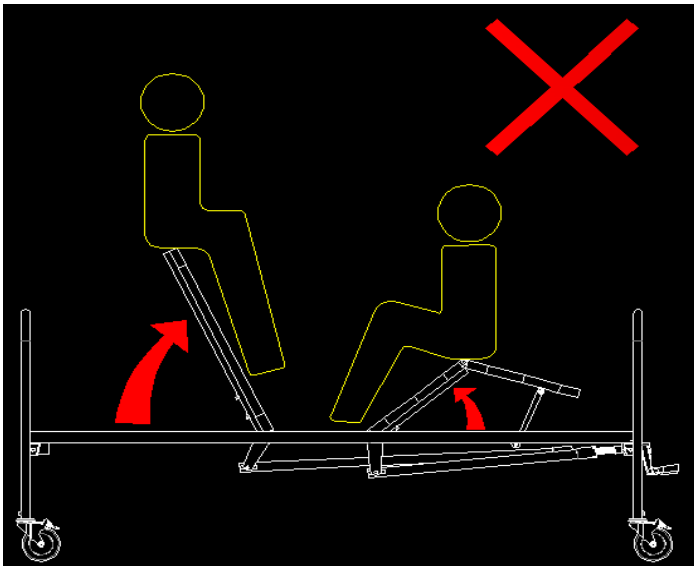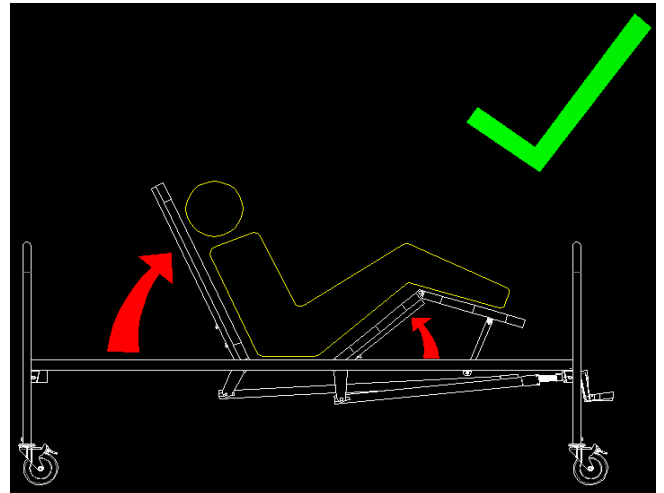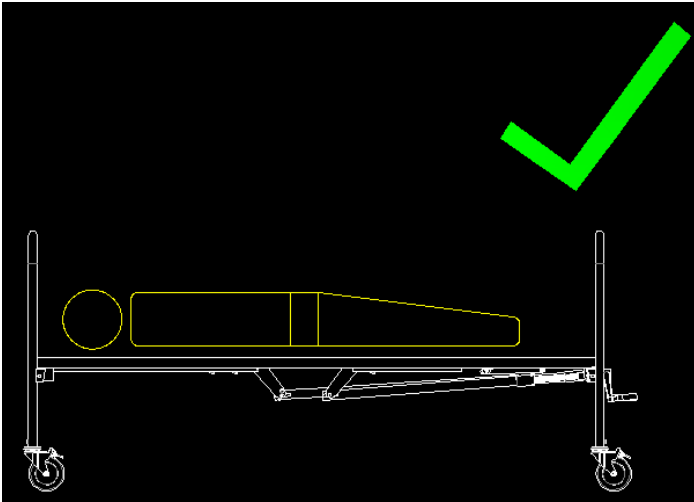DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity for any reason.
- When lowering the head or foot section, make sure that nobody inserts hands, legs or other body parts under the bed base
- Make sure that the floor surface supporting the bed is flat.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
2. In case of damage do not use the product and contact your dealer for further instructions.
3. Check, if the product is properly assembled and all screws are secure and well screwed on.
4. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

MATTRESS SELECTION

Use mattresses or anti-decubitus mattresses according to the prescription of the treating Physician. The size of the mattress should not be larger than the frame of the bed. All MOBIAK mattresses and anti-decubitus are suitable for Jasmin & Elegant beds.




ASSEMBLING:

NOTE: The illustrations for the assembly and adjustments refer to the bed with the 2 cranks. However, they can also be applied to the bed with a single crank. The package does NOT contain tools for assembly procedure.

PACKAGE CONTENTS:

	DESCRIPTION	QTY
1	Main Bed Frame	1
2	Crank for Backrest	1
3	Crank for Legrest (only for item 0805421)	1
4	Brackets for Backrest Crank	1
5	Brackets for Legrest Crank (only for the 0805421)	1
6	Side Bed Frame (Head and Foot)	2
7	Wooden panel for side frame	2
SCREWS KIT		
A	Screws to fix the crank to the bed frame	2 pcs for 0805420 4 pcs for 0805421
B	Screws to fix the side frames (Head and Foot)	4
C	Pins & split pins for fixing the brackets to the bed frame and the crank tubes	3 pcs for 0805420 5 pcs for 0805421

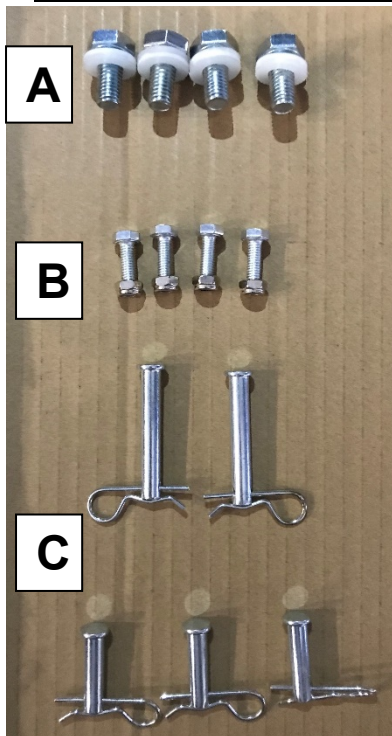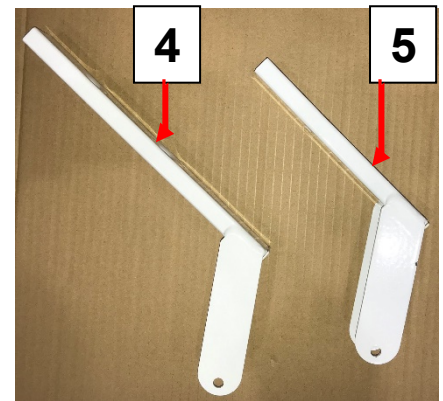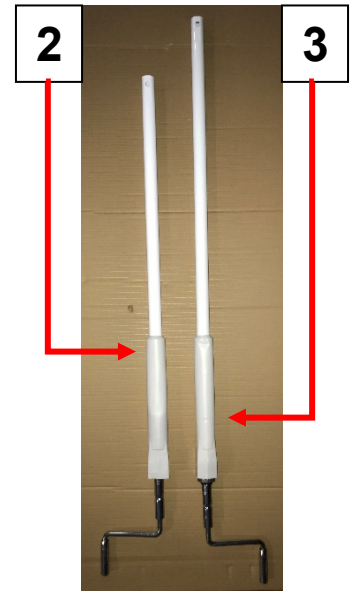FOLLOW THE BELLOW STEPS:

1. Open the package and remove all parts of the product, except the main frame.
2. Check that the content Complies with the above content list.
3. Place the Backrest Crank bracket "2" (and the Legrest Crank bracket "3", if you have item 0805421).
4. Secure with the Pins & split pins "C".


As the following picture show: # 1 to # 6

FOLLOW THE BELLOW STEPS:

5. Place the Backrest Crank (and the Foot Crank, if you have item 0805421) into the slots on the edge of the bed frame and screw & secure with the "A" screws.
6. Secure with the Pins & split pins "C".on the other end of the Crank on the bracket "4" or even "5"

As the following picture show:
#7 to #10

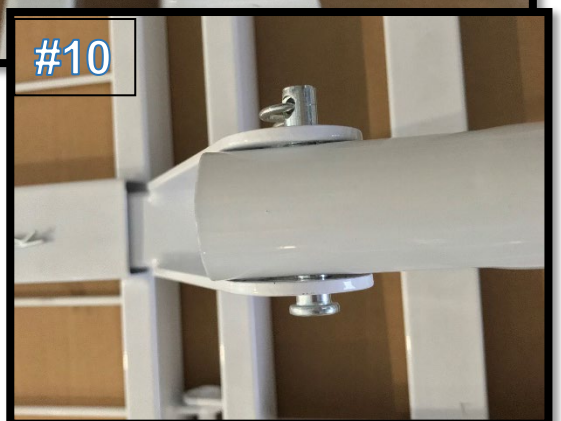
FOLLOW THE BELLOW STEPS:

9. Turn the bed on the right side and join the male sockets that are on the bed legs with the female that are on the Side Frame "6" (headboard and footrest).

10. Screw & Secure with screws "B".

As the following picture show: #11 to #13

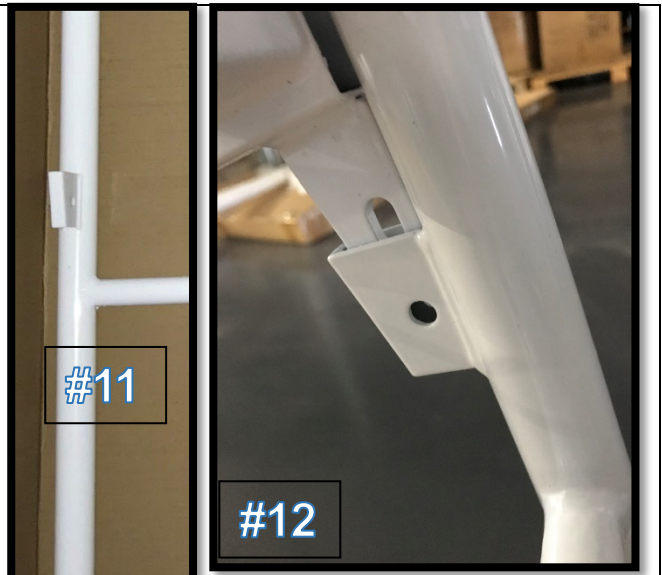FOLLOW THE BELLOW STEPS:


7. Place the Wooden Side Frames "7" in the special iron slots of the Side Frame of the Bed "6" (headboard and footrest).

8. And press lightly down to secure.

As the following picture show: #14 to #16

The Bed is assembled and ready for use.

DISASSEMBLY: Follow the steps described above in reverse order.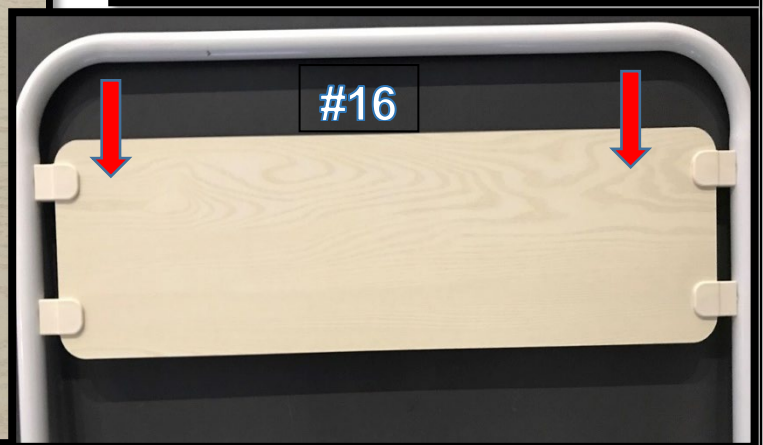

USE OF CRANK:

- i. Pull the handle out.
- ii. Rotate the handle clockwise to raise the section and counterclockwise to lower it
- iii. Tap the Handle inward after adjusting the slop to avoid any injuries.

Offered separately and optionally:

- 3. Wheel 4" with brake, ref:**0805422**
- 4. Monkey Pole with plastic handle ref.:**0805423**

To assemble the wheel with the bed



- 1) Remove the plastic cap from the foot of the bed.
- 2) Insert the wheel socket into the bed tube.
- 3) And screw well to secure the wheel.

Brake activation: push the brake activation lever down with your foot.

Brake deactivation: push the brake deactivation lever down with your foot.

To place the Monkey Pole with plastic handle on the bed:

- 1) Assemble the Monkey Pole that comes in three parts.
- 2) Slide it into the special slot that are on the left or right of the head section.

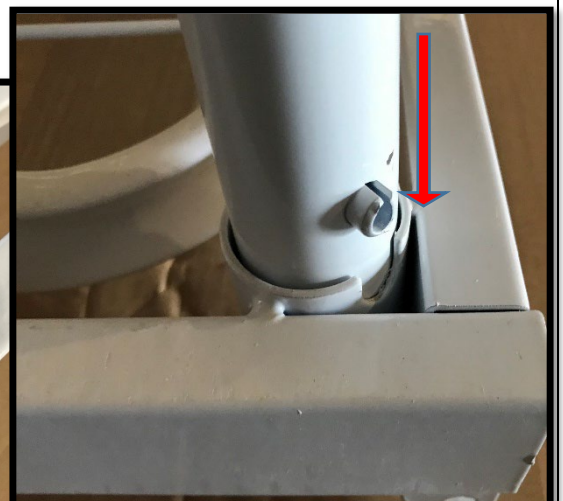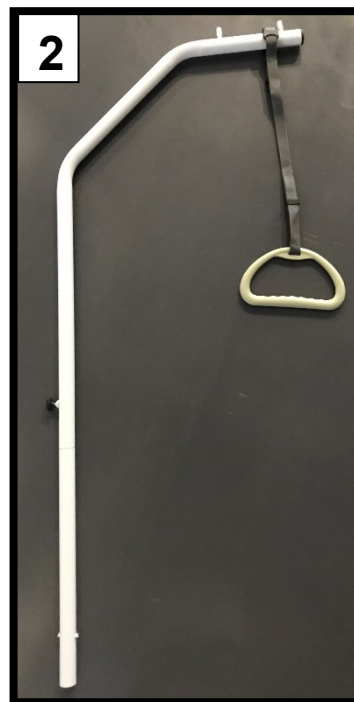CLEANING AND DISINFECTION:

Use only a wet cloth and mild soap on surfaces where they need cleaning.

Then dry thoroughly before you use the product.

DO NOT leave the product wet.

Do not use chemical cleaners to clean the frame. This could cause damage to the surface.

If you need to disinfect the product, use a common, mild disinfectant.

Protect the item from scratches, cuts and punctures.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, brakes, wood or plastic parts.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

WARRANTY FORM:

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

MOBIAC S.A.
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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