


Patient access to Expert Info menu

Should you wish, e.g. on the phone, to query the FW version of a device or a device counter other than N3 (which appears directly on startup), you can explain direct access to the patient using the Expert Info menu. This access only opens the Info menu so that the patient is unable to make any changes to the configuration. Please also use this direct access when you wish to give the patient an insight into his/her therapy results such as AHI and leaks.

Condition: Device is on standby.

1. Press Menu key .
Patient menu opens.

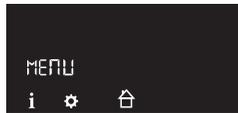

- Hold down softSTART key  for a while.
Expert Info menu opens.

Navigating in the menus

Function key	Description
	Scroll forward through menu
	Scroll back through menu
	Increase value / change displayed time period
	Reduce value / change displayed time period
	Confirm value
	Discard value
	Exit menu. Back to start screen.

CE 0197

 **Löwenstein Medical
Technology GmbH + Co. KG**
Kronsaalweg 40
22525 Hamburg, Germany
T: +49 40 54702-0
F: +49 40 54702-461
www.loewensteinmedical.com

mbh-10031
WM 68221c EN · 04/2023



Short Operating Instructions for Medical Health Professionals

prisma SMART/SOFT

prisma SMART/SOFT plus/max



- 1  key
- 2  key
- 3  On / off key
- 4  Menu key
- 5  softSTART key



This document does not replace the instructions for use. Full information can be found in the instructions for use.

LÖWENSTEIN
medical

LÖWENSTEIN
medical

Expert selection menu

- Hold down the menu key  and softSTART key  simultaneously.

The Expert menu opens.



To call up the selection menu: Press the  (+ key).

Parameter	Description
Mode 	CPAP / APAP
CPAP mode: CPAP pressure 	You can select CPAP pressure here.
APAP mode: Minimum APAP pressure 	You can select minimum APAP pressure here.
	Maximum APAP pressure 
APAP dynamics  	You have the option of choosing between two different settings for APAP control: <i>Std (Standard)</i> : Known APAP algorithm: Fast, reliable therapy for apneas and hypopnoeas is offered here. The device responds to snoring, flattening and RERA. <i>dYn (dynamic)</i> : A stronger and faster response to mild obstructive events such as snoring and flattening. Respiration is normalized to a greater extent.
softPAP level 	softPAP easing of exhalation is suitable for patients who find it unpleasant to exhale against a high pressure. The device temporarily reduces therapy pressure in stages 1 (slight easing of exhalation) and 2 (standard easing of exhalation) before the transition to exhalation. Switch softPAP to  to deactivate this function.

Parameter	Description
softPAP lock 	You can choose here whether the patient can set softPAP him-/herself  or whether this function is locked for the patient  .
autoSTART 	If autoSTART is activated () , the device can be switched on by a breath (> 0.5 hPa) being taken into the mask and switches off automatically after 5 seconds with the mask being removed. Switch autoSTART to  to deactivate this function.
softSTART time 	Here you can set the time for which ventilation pressure rises to therapy pressure during the softSTART.
softSTART pressure 	You can select softSTART pressure here. This parameter is not available when softSTART time is  .
Tube type 	Here you can choose the diameter of the tube type used.
Tube type lock 	You can choose here whether the patient can set tube type him-/herself  or whether this function is locked for the patient  .
Time 	You can set the time of day here.
Format time display 	Here you can choose whether the time is to be displayed in the form 0 – 24 h () or 0 – 12 h ()
Blue 	Here you can permanently deactivate/activate the Bluetooth function for the patient. For plus/max devices only.
5rCH  diAG 	The fields alternate. This is where to activate/deactivate diagnosis mode. The period of activation can be specified as 1d (1 day) to 7d (7 days). The device searches for a diagnosis device. prisma APP will not be able to find the therapy device whilst diagnosis mode is activated.
Cellular 	Here you can permanently deactivate/activate the internal modem for the patient. For max devices only.
Delete data 	Here you can delete the counters N1, N2, N3, as well as the statistics values of the device. The device configuration set for the patient remains unchanged.

Expert Info menu

- Hold down the menu key  and softSTART key  simultaneously.

The Expert menu opens.



To call up the info menu: Press the  (- key).

Parameter	Description
	Usage time (N3)
	The time specified is the average therapy time (without artefacts and mask open) per day in the specified period*
	Mean total AHI in the selected period*
	Leak median in the selected period*
	90% pressure percentile in the selected period*
	Cheyne-Stokes respiration*
	Firmware version. X stands for the main version number.
	Therapy hours without artefacts and mask open
	Therapy hours with humidifier (without artefacts and mask open)
	Usage time incl. artefacts and mask open
	Blower runtime

*With these parameters you can use the  key and the  key to display additional periods of time:

	Last day
	Last 7 days
	Last 28 days
	Last 182 days
	Last 366 days

Data are displayed only if they really are present in the device. Each therapy day begins and ends at 12 noon. Data recorded from midnight to 12 noon are assigned to the previous calendar day.