

Tear Here / Άνοιγμα εδώ

**Well Lead**®



0123

code: 0803118 / R01A02021F

## Μάσκα Οξυγόνου Ενηλίκων με σωλήνα 2,10 μέτρων Oxygen Mask for Adults with tube 2,10 meters

### Instructions of use / Οδηγίες Χρήσης

**EN** - Place the mask on a patient and ensure a comfortable fit. Adjust the nose clip on the portion of the mask, mold it to fit the face. This precludes leakage of oxygen into the corners of the eyes. If the patient feels respiration insufficient, please enlarge the exhalation ports with knife or scissors.

**GR** - Τοποθετήστε τη μάσκα στον ασθενή και εξασφαλίστε άνετη εφαρμογή. Ρυθμίστε το κλιπ μύτης στο τμήμα της μάσκας, διαμορφώστε το ώστε να ταιριάζει στο πρόσωπο. Αυτό αποκλείει τη διαρροή οξυγόνου στις γωνίες των ματιών. Εάν ο ασθενής αισθάνεται ότι η αναπνοή είναι ανεπαρκής, διευρύνετε τις θύρες εκπνοής με μαχαίρι ή ψαλίδι.

### Caution / Προσοχή

**EN** - For single use. Discard after use.

Do not store in direct sunlight, at extreme temperatures or in high humidity.

Store in dry conditions.

Do not use if package is open or damaged.

**GR** - Για ατομική χρήση. Απορρίψτε μετά τη χρήση.

Μην το αποθηκεύετε σε άμεσο ηλιακό φως, ακραίες θερμοκρασίες ή σε υψηλή υγρασία.

Αποθηκεύστε στο σε ξηρές συνθήκες.

Μην χρησιμοποιείται αν η συσκευασία είναι ανοιχτή ή έχει υποστεί ζημιά.

DISTROBUTOR:



96-98 str. M. Mpotsari  
Chania - Crete - Greece

Tel: +30 28210 63222

Web Site: [www.mobiakcare.gr](http://www.mobiakcare.gr)

Manufacturer:

Well Lead Medical Co.,Ltd.

C-4 Jinhui Industrial Estate, Hualong

511434 Panyu, Guangzhou

PEOPLE'S REPUBLIC OF CHINA

EC-Representative:

Shanghai International Holding

Corp. GmbH (Europe)

Eiffestraße 80

20537 Hamburg

GERMANY

LOT :

⏏ :

⏰ :

1 PC

**Well Lead**®

code: 0803118 / R01A02021F

**Μάσκα Οξυγόνου Ενηλίκων με σωλήνα 2,10 μέτρων  
Oxygen Mask for Adults with tube 2,10 meters**

DISTROBUTOR:

**MOBIAK**  
MEDICAL • GASES • FIREPROTECTING 001 077  
96-98 str. M. Mpotsari  
Chania - Crete - Greece

Tel: +30 28210 63222  
Web Site: [www.mobiakcare.gr](http://www.mobiakcare.gr)

Manufacturer:  
Well Lead Medical Co., Ltd.  
C-4 Jinhui Industrial Estate, Hualong  
511434 Panyu, Guangzhou  
PEOPLE'S REPUBLIC OF CHINA

EC-Representative:  
Shanghai International Holding  
Corp. GmbH (Europe)  
Eiffestraße 80  
20537 Hamburg  
GERMANY



0123

**0803118**

Ref: **R01A02021F**

**LOT**

Type: Oxygen Mask

Size: Adult

QTY 50 pcs

G.W: Kg

(LxWxH): 48x36x28cm

N.W: Kg

